# Rigel SafeTest 60 Electrical Safety Analyzer

## Quick Start Guide

### Warnings and Cautions

- Warning of electrical danger!**  
Indicates instructions must be followed to avoid danger to persons.
  - Important, follow the documentation!**  
This symbol indicates that the operating instructions must be adhered to in order to avoid danger.
  - Warning**  
During many of the leakage tests the PROTECTIVE EARTH/GROUND CONDUCTOR to the appliance under test is interrupted. For this reason additional means of protection are required in order to comply with the relevant safety standards.
  - Safety**  
Do not connect any probe combination to the maximum permitted voltage of 30VAC/DC with respect to earth/ground potential when performing non-power tests. Danger of electric shock.
- It must be assumed that safe operation is no longer possible:**
- If the instrument or leads show any sign of damage
  - If the instrument does not function
  - After long periods of storage under adverse environmental conditions

### Introduction

Welcome to your new SafeTest 60 hand-held Electrical Safety Analyzer. This Quick Start Guide tells you how to set up your SafeTest 60 and make use of its key features.

### 1 Getting to know your SafeTest 60

**SafeTest 60**

- Mains/line, START, STOP, REVERSED keys
- Function keys F1 - F5
- Single fault OPEN EARTH/GROUND - OPEN NEUTRAL keys
- EUT socket, 90-264VAC, 48-64Hz
- Point-to-point auxiliary probe
- Earth/ground bond and enclosure leakage
- Mains/line inlet 90-264VAC, 48-64Hz

**What's in the box?**

- Rigel SafeTest 60
- Detachable 2m mains lead
- Earth/ground bond test probe with clip
- Calibration Certificate
- Carry Case

**NOTICE**  
If any items are missing on receipt of your new unit, please contact your equipment supplier immediately.

### 2 Getting started

In this section we will identify some of the basic functions of the Rigel SafeTest 60 and how to get you started.

#### Switch on

To switch on the Rigel SafeTest 60, please insert the power cord to the power inlet. The SafeTest 60 will automatically power up.

### 3 Perform electrical safety tests

From the HOME screen, select the required test;

- Ω** To select EARTH/GROUND BOND TESTING
- μA** To select LEAKAGE TESTING
- MΩ** To select INSULATION TESTING
- PPT** To select POINT TO POINT TESTING
- ⚙️** To select SETTINGS menu (change LANGUAGE and TEST STANDARD)

### 3a Settings menu

Press **⚙️** from the HOME screen to enter SETTINGS menu;

- Press the **↔️** key to change to the required language
- Press the **⬇️** key to change to the required test standard
- Press the **i** key to view the firmware and hardware information
- Press the **✓** key to confirm and return to the HOME menu

### Need help?

This Quick Start Guide is designed to be used in conjunction with the full SafeTest 60 instructional manual, available as a download from [rigelmedical.com/rigel-downloads](http://rigelmedical.com/rigel-downloads)

For technical issues please visit [rigelmedical.com](http://rigelmedical.com) and view the Application Notes in the Downloads tab. Application Notes are regularly updated and will most likely cover any common issues.

For further assistance please contact your Rigel Medical supplier or use the following contact details to speak to a member of the Rigel Medical team:

**Sales and Delivery enquiries**  
Email: [sales@rigelmedical.com](mailto:sales@rigelmedical.com)  
Tel: +44 (0) 191 587 8730 Fax: +44 (0) 191 586 0227  
USA office: +1 813 886 2775

**Technical enquiries**  
Email: [support@rigelmedical.com](mailto:support@rigelmedical.com)  
Tel: +44 (0) 191 587 8701 USA office: +1 813 886 2775

**Service, Calibration and Repair**  
Email: [service@calibrationhouse.com](mailto:service@calibrationhouse.com)  
Tel: +44 (0) 191 587 8739 Fax: +44 (0) 191 518 4666  
USA office: +1 813 886 2775

Remember to activate your 2 year warranty\* by registering your Rigel SafeTest 60 with Rigel Medical

Go to [rigelmedical.com/register-product](http://rigelmedical.com/register-product)  
\*Terms and Conditions apply. See website for details.

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