


QUICK SETUP
GUIDE

SECULIFE IS



The SECULIFE IS simplifies quality assurance for diagnostic monitors and light boxes in medical applications in accordance with DIN V 6868-57. At the same time, it can be used to monitor room classes for medical tasks which are specified in the forthcoming DIN 6868-157 standard. Using the SECULIFE IS extends the interval specified in the standards for constancy testing of image displays to 6 month with regard to veiling luminance and maximum contrast.

SECULIFE IS

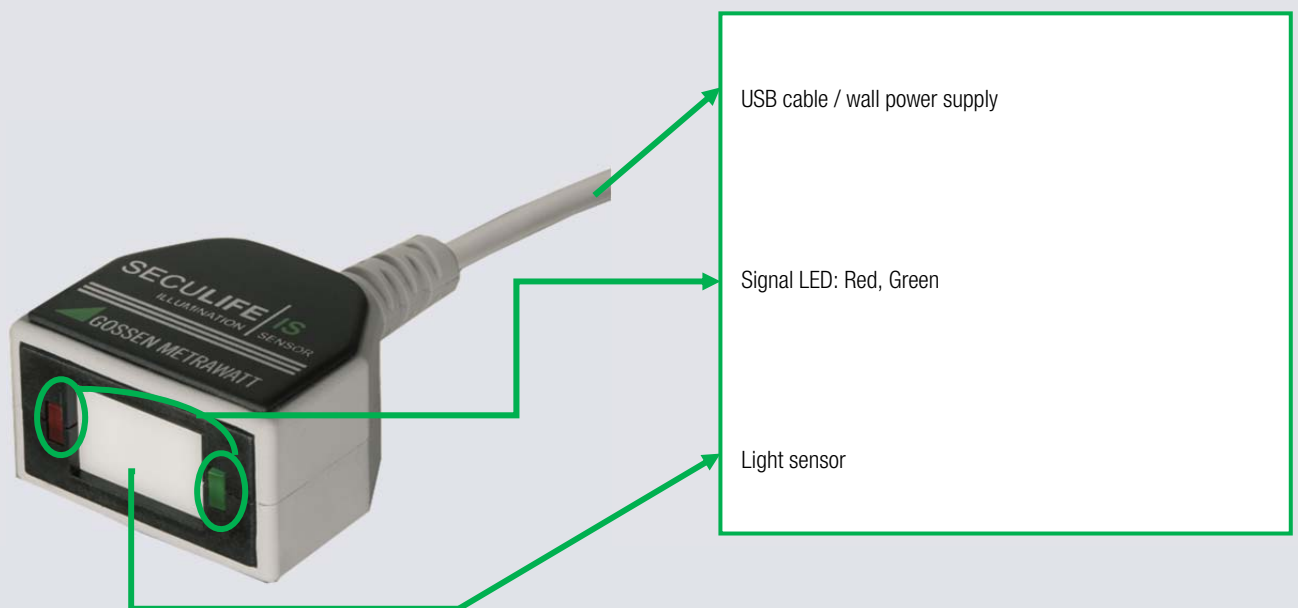

Initial Start-Up

Initial start-up of the SECULIFE IS is quick and easy:

The instrument is connected with the included power pack, or to a USB port at the measuring PC, and is attached to the monitor screen.


The SECULIFE IS is attached to the front edge of the monitor with the help of the double-faced tape at the bottom. The front of the SECULIFE IS must be flush mounted to the front of the monitor. Please note that the SECULIFE IS can no longer be removed after attaching it to your monitor.

WARNING! Forcible removal of the monitoring unit will destroy the base plate of the SECULIFE IS!

- 1) Attach SECULIFE IS to the monitor
- 2) Via USB cable for power supply: connecting it to an available USB port or the included mains power pack
- 3) Pay attention to LEDs



GMC INSTRUMENTS

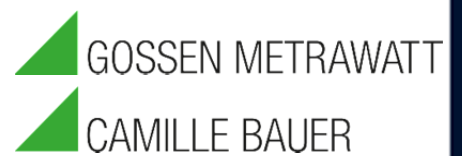

GMC-I Messtechnik GmbH

Südwestpark 15 □ 90449 Nürnberg □ Germany

TEL +49 911 8602-111 □ FAX +49 911 8602-777

www.gossenmetrawatt.com □ info@gossenmetrawatt.com